



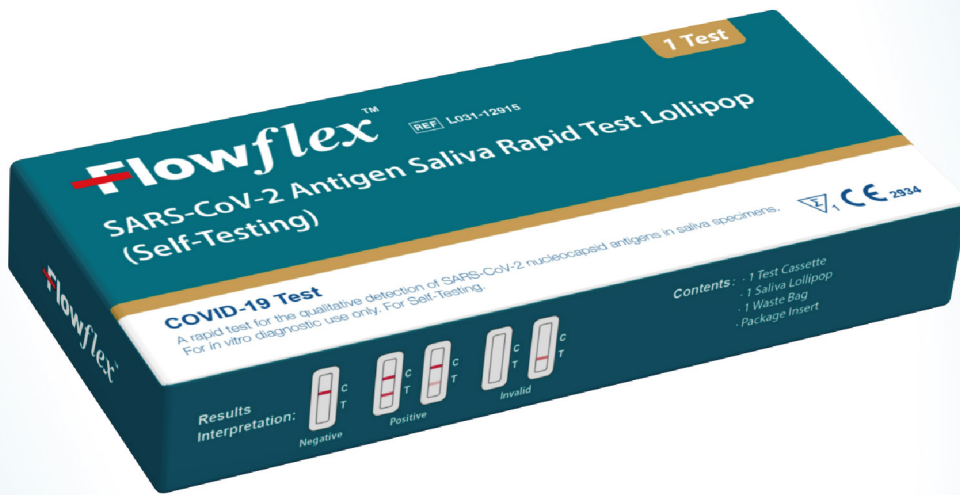

# SARS-CoV-2

## Antigen Saliva Rapid Test Lollipop (Self-Testing)

A rapid test for the detection of SARS-CoV-2 nucleocapsid antigens in saliva specimens.  
For in vitro diagnostic use only. For self-testing.

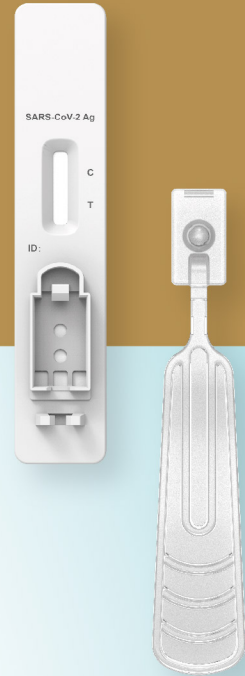

World Leader in Rapid Test Industry



Fast



Accurate



Easy to Use



Reliable



CE Marked



# Flowflex SARS-CoV-2 Antigen Saliva Rapid Test Lollipop (Self-Testing)

The SARS-CoV-2 Antigen Saliva Rapid Test Lollipop (Self-Testing) is a lateral flow test for the qualitative detection of the nucleocapsid antigen from SARS-CoV-2 in saliva specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- **Specimen:** Saliva specimens
- **Test Time:** Results at 15 min.
- **Shelf life:** 24 months
- **Storage temperature:** 2-30°C
- **Accuracy:** 99.6%
- **Sensitivity:** 100.0%
- **Specificity:** 99.6%



1T




25T

## Test Procedure

### 1. Test Preparation


#### A. You should have:

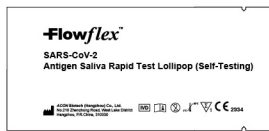

Test cassette



Saliva Lollipop

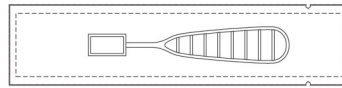

Package Insert




Waste Bag


#### B. Wash or sanitize your hands.



#### C. Read the instructions.

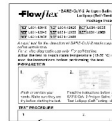


#### D. Check the expiration date.

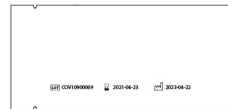


#### E. Open the pouch.




#### F. Check the cassette

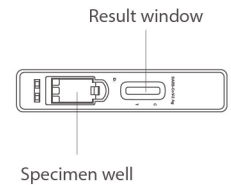

### 2. Test Steps

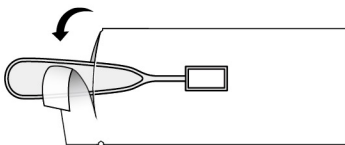


1 Open the saliva lollipop packaging at stick end.

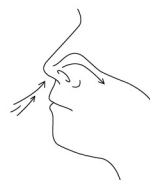


2 Drain mucus from the back of the nose and throat at least 3 times.



3 Take a deep breath in and cough hard, do not spit.

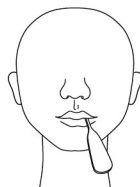


4 Insert a saliva lollipop into the mouth at least 30 seconds.

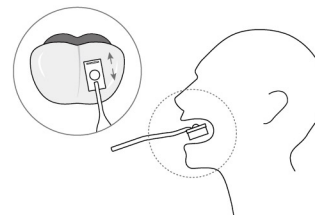

5 Wipe both **upper and down surface** of the tongue.

### 3. Running the Test

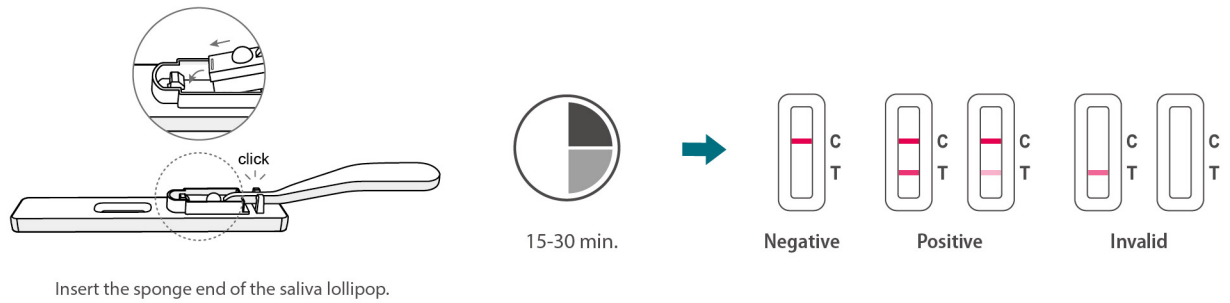

### Ordering Information

Product Name	Catalog No.	Format	Specimen	Package
<b>Flowflex SARS-CoV-2 Antigen Saliva Rapid Test Lollipop (Self-Testing)</b>	L031-12915 ✓	Cassette	saliva specimens	1 Test/Kit
	L031-12925 ✓	Cassette	saliva specimens	2 Tests/Kit
	L031-12935 ✓	Cassette	saliva specimens	3 Tests/Kit
	L031-12945 ✓	Cassette	saliva specimens	5 Tests/Kit
	L031-12955 ✓	Cassette	saliva specimens	7 Tests/Kit
	L031-12965 ✓	Cassette	saliva specimens	10 Tests/Kit
	L031-12975 ✓	Cassette	saliva specimens	20 Tests/Kit
	L031-12985 ✓	Cassette	saliva specimens	25 Tests/Kit

✓ CE Marked



**ACON Biotech (Hangzhou) Co., Ltd.**  
 No.210 Zhenzhong Road, West Lake District, Hangzhou, P.R.China, 310030  
 Tel: +86-571-8118 9706 / Email: information@aconlab.com.cn  
[www.aconbio.com](http://www.aconbio.com)